
Cat. No. D3062 - Protocol



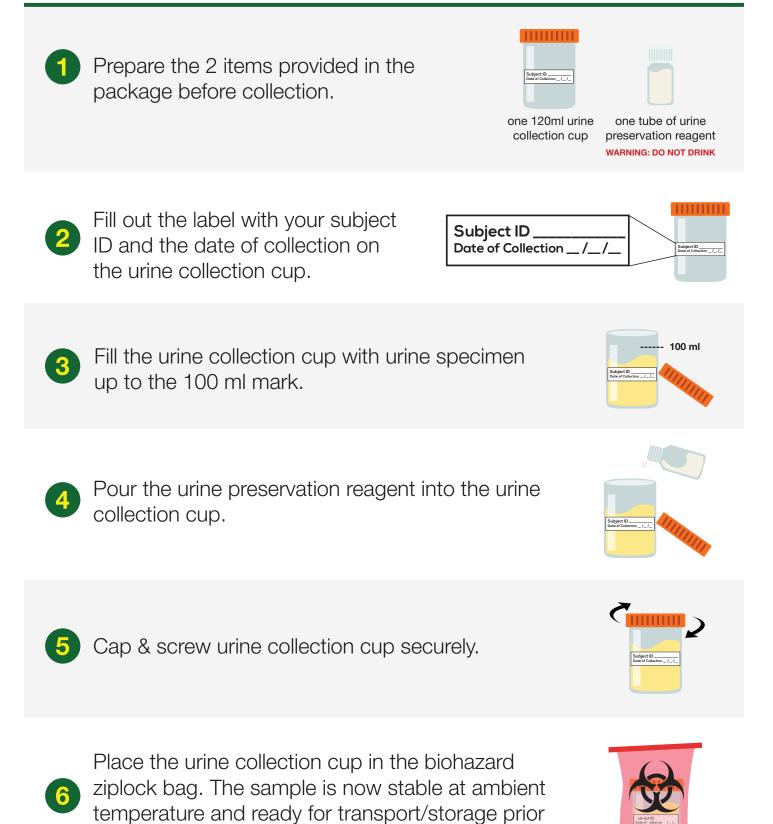

to purification of DNA and/or RNA.



## Product Description

The Urine Collection Kit w/ Urine Conditioning Buffer ensures urine sample stability during storage/transport and ambient temperatures without a need for refrigeration or specialized equipment. Urine Conditioning Buffer effectively inhibits microbial growth in urine samples and the urine nucleic acid content is preserved at ambient temperature for up to 1 month. Urine Conditioning Buffer also facilitates pelleting of urine nucleic acids from large volume urine samples for the subsequent nucleic acids purification.

## **DNA/RNA Extraction**

To proceed with urine DNA/RNA purification, mix urine thoroughly, and centrifuge at 3,000 x g for 15 minutes to form a urine pellet. Purify urine nucleic acid directly with Zymo Research's DNA or RNA purification kits.

Sample Type	DNA	RNA	Both
Urine	Quick-DNA™ Urine Kit	Quick-RNA™ (Plus)	ZR-Duet™ DNA/RNA (Plus)
	D3061	R1057, R1058	R7003

Note: Urine pelleted utilizing Urine Conditioning Buffer™ can also be processed with other commercially available DNA and RNA purification kits.

## Ordering Information

Product Description	Catalog No.	Kit Size
Urine Collection Kit w/ Urine Conditioning Buffer™	D3062	10 pack

For Individual Sale	Catalog No.	Kit Size
Urine Conditioning Buffer™	D3061-1-8 D3061-1-140	8 ml 140 ml
Urine Collection Cup (120 ml)	D3062-1	10 pack